# SOPIRA® Carpule®

## Sterile Single-Use Dental Needles


- DE** Gebrauchsanweisung
- GB** Instructions for use
- FR** Mode d'emploi
- ES** Instrucciones de uso
- IT** Istruzioni per l'uso
- PT** Instruções de uso
- NL** Gebruiksaanwijzing
- SE** Bruksanvisning
- DK** Brugervejledning
- NO** Bruksanvisning
- FI** Käyttöohjeet

### Heraeus

Manufactured by:  
Heraeus Kulzer GmbH  
Grüner Weg 11  
63450 Hanau (Germany)  
info.dent@heraeus.com  
www.heraeus-dental.com

99000887/01

CE 0197

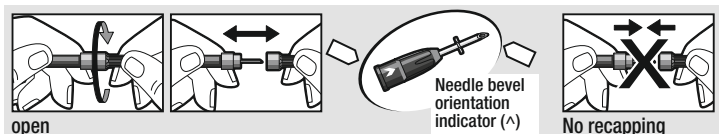## SOPIRA® Carpule®

### Instructions for use (GB)

- 100 sterile, individually packed disposable dental needles for single use
  - Silicone coated for easier penetration and sterile.
  - Plastic hub with thread and needle bevel orientation indicator for optimum injection direction and reduced insertion pain.
  - Thin-wall SOPIRA Carpule Free Flow cannula for low injection pressure.

#### Use

1. Protect needles against direct sunlight, high temperature and moisture.
2. Only use needles if the seal on the packaging is intact.
3. Needles may be used with all commercial syringes and syringe systems in common use in dental anaesthesia. Please choose appropriate length and thickness of needles for specific indication in dental local anaesthesia.
4. Before use, unscrew the short, transparent protective cap (removing the long protective cap just before injection).
5. Insert the short needle into the syringe and screw onto the thread of the syringe with the long protective cap on. The needle is properly attached to the syringe when the two parts of the thread (on the syringe and needle) fit together flush (i.e. with no protrusions).
6. Only remove the long protective cap just before injection.
7. Do not bend the needles.
8. Each needle may only be used for one single patient.
9. Avoid all contact with the used needle (risk of infection, do not manually recap!). To avoid cross infection, items intended for single use must be disposed of immediately after initial use.
10. Dispose of needles in special safe containers (puncture-resistant/lockable), which are available commercially.
11. Needles should be collected and protected against human contact (i.e. in a suitable protective container) and then disposed of in accordance with valid regulations.
12. Only for professional use.

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## SOPIRA Carpule Needles

| Available with metric thread | Indication        | Available as Free-Flow | Available as Blister | Hub colour (indicating diameter) | Label colour (indicating length) * |
|------------------------------|-------------------|------------------------|----------------------|----------------------------------|------------------------------------|
| 0.4 x 08 mm 27 G             | Intraseptalis     |                        |                      | grey                             | violet                             |
| 0.4 x 08 mm 27 G short bevel | Intraligamentalis |                        | Blistered only       | grey                             | violet                             |
| 0.4 x 25 mm 27 G             | Infiltration      | ✓                      | ✓                    | grey                             | orange                             |
| 0.4 x 35 mm 27 G             | Mandibularis      |                        | Blistered only       | grey                             | red                                |
| 0.4 x 38 mm 27 G             | Mandibularis      | ✓                      |                      | grey                             | dark red                           |
| 0.3 x 12 mm 30 G short bevel | Intraligamentalis | ✓                      | ✓                    | yellow                           | blue                               |
| 0.3 x 12 mm 30 G             | Infiltration      |                        |                      | yellow                           | blue                               |
| 0.3 x 16 mm 30 G short bevel | Intraligamentalis | ✓                      | ✓                    | yellow                           | green                              |
| 0.3 x 16 mm 30 G             | Infiltration      |                        | ✓                    | yellow                           | green                              |
| 0.3 x 21 mm 30 G             | Infiltration      |                        | ✓                    | yellow                           | yellow                             |
| 0.3 x 25 mm 30 G             | Infiltration      | ✓                      | ✓                    | yellow                           | orange                             |
| Available with inch thread   |                   |                        |                      |                                  |                                    |
| 0.4 x 35 mm 27 G             | Mandibularis      |                        |                      | grey                             | red                                |
| 0.3 x 12 mm 30 G short bevel | Intraligamentalis |                        |                      | yellow                           | blue                               |
| 0.3 x 16 mm 30 G short bevel | Intraligamentalis |                        |                      | yellow                           | green                              |
| 0.3 x 21 mm 30 G             | Infiltration      |                        |                      | yellow                           | yellow                             |

\* In addition to the hub colour indicating the diameter, the coloured paper label is used to code the length of the needles. Please find further information on [www.heraeus-sopira.com](http://www.heraeus-sopira.com)

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