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[www.prodonto.en/d/siliconi.pdf]

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Instruction for use EN

[www.prodonto.en/d/siliconi.pdf]

Non-powdery alginates for impressions.

INSTRUCTIONS FOR USE

PrintØ PRO FAST: alginate with a creamy consistency, does not compress the tissues, particularly suitable for preliminary impressions of edentulous jaws. Aroma: Mint

PrintØ PRO CHROMO: high consistency chromatic alginate. The chromatic variation accompanies the different preparation phases: purple during mixing, grey during the working phase and positioning on the impression tray, green during the dwell time in the oral cavity until hardening.

Recommended for general clinical practice. The chromatic change associated with a pleasant aroma makes it particularly suitable for use in paediatric dentistry. Aroma: Raspberry

DOSAGE AND MIXING

Open the bag (fig. 1) and pour the alginate into an airtight container (fig. 2). It is advisable to keep the lot number and expiration date of the product, cutting out the information from the bag. Shake 2 or 3 times to facilitate the dispersion of the powder (fig. 3) and proceed with the dosage: with the aid of the measuring spoon, take the alginate without compressing the powder, level or remove the excess alginate by using the flat part of the spatula on the spoon (fig. 4), and pour it into a rubber cup. Repeat the operation until the desired quantity is reached. For each tablespoon of powder (9 g), pour 1/3 of a measuring cup of water (18 ml). E.g., for 2 measures of powder (18 g) sufficient for a higher middle spoon, pour 2/3 of a measuring spoon of water (36 ml). (fig. 5).

Close the container and store it at 5-27°C / 41-80°F (fig. 6).

Add the water to the powder to start mixing according to the times recommended in the table (fig. 7), place the material on the impression tray (fig. 8), and insert it into the oral cavity before the end of the

processing time (fig. 9).

Wait for the material to harden, observing the recommended dwelling times in the oral cavity. In chromatic alginates, processing times are easily recognized by colour changes, as described above.

TECHNICAL DATA

TIMES OF CLINICAL USE*	PrintØ PRO FAST	PrintØ PRO CHROMO
Mixing time	30"	45"
Processing time	1'10"	1'35"
Dwell time in the oral cavity	1'00"	1'00"
Setting time	2'10"	2'35"

* Clinical use times are intended from the start of mixing with deionized water at 23°C / 73°F. Cold water delays the hardening, hot water accelerates it. Variations in water hardness can modify setting times. Corresponds to ISO 21563 specifications.

CLEANSING AND DISINFECTION OF THE DENTAL IMPRESSION

After taking the impression, rinse thoroughly to eliminate any residues and traces of saliva (fig. 10), remove excess water and disinfect immediately. To disinfect the impression, follow the instructions specified by the disinfectant manufacturer (fig. 11).

IMPRESSION CASTING - RECOMMENDED PLASTERS

It is advisable to cast the impression immediately with type 3 (e.g., Elite Model) or type 4 (e.g., Elite Rock) plaster (fig. 12).

PRESERVATION OF THE IMPRESSION

If it is not possible to immediately pour the plaster, remove excess water and store the impression in a hermetically sealed bag at room temperature (23°C / 73°F). Under these conditions, the casting of the plaster can be deferred for up to 48 hours after taking the impression (fig. 13).

IMPORTANT REMARKS: Any advice provided verbally, in writing, or through demonstrations, on the use of our products, is based on the current state of dental technology and our know-how. The advice is to be considered as non-binding information, also in relation to any rights of third parties and does not exempt the professional operator from personally checking whether the product is suitable for the intended application. The use and application by the professional operator take place without the possibility of monitoring by the Company and therefore are subject to the responsibility of the professional operator. Any liability for damage is limited to the value of the goods supplied by the Company and used by the professional operator. The product contains diatomaceous earth treated with a wetting agent to reduce the formation of breathable dust.

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Astidental S.p.A.